




NewCare Bed

QUICK REFERENCE SHEET

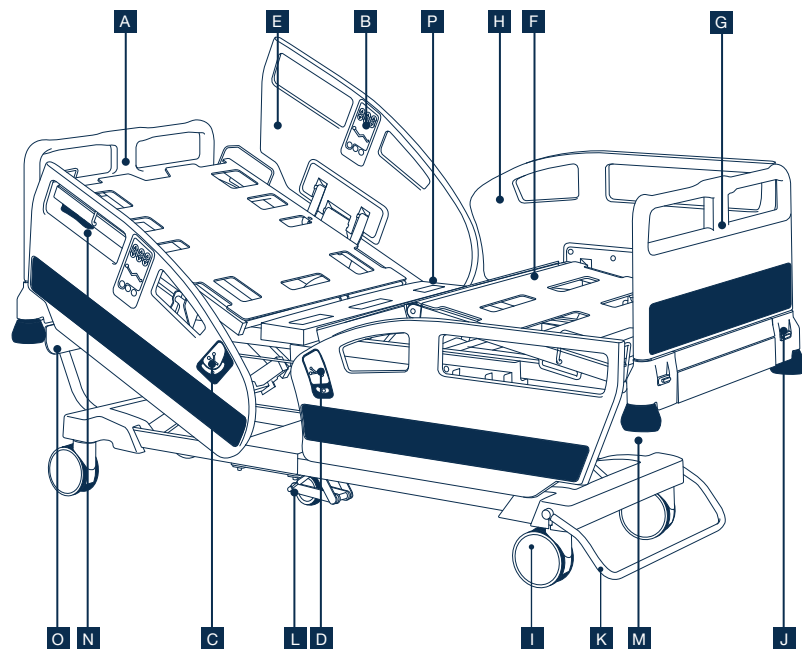

BED'S COMPONENTS

A	Removable curved headboard	I	Double wheels Ø15 cm.
B	Membrane side rails control	J	Headboards and footboards Unlocking
C	Backrest tilting indicator	K	Centralised braking system
D	Trend / Reverse Trend tilting indicator	L	5 th wheel
E	Backrest side rails	M	Protection discs
F	HPL mattress platform	N	Emergency CPR
G	Removable curved footboard	O	Anchored headboard
H	Legrest side rail	P	Sheet steel central section

www.pardo.es

TECHNICAL SERVICE CENTRE

(+34) 976 300 033

OTHER FUNCTIONS

Side Rails (1):


- Raising the rail: Pull the rail upwards until it is locked in the highest position.
- Lowering the rail: Pull the handle found underneath the rail outwards. The rail will automatically lower.

Emergency CPR (2):


- Activate either of the two levers located under the headrest module, accompanying it in turn as it lowers.

Centralised braking system (3):



- Braking the bed: Pedal in low position.
- Moving the bed: Pedal in the middle position.
- Activating the 5th or steering wheel: Pedal in raised position.

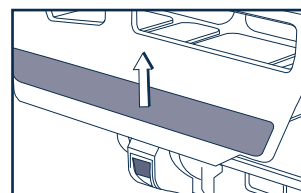


Rising the rails

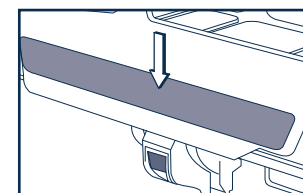


Lowering the rails

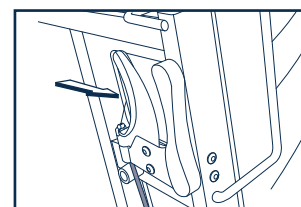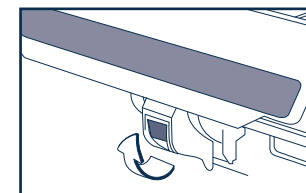


Emergency CPR

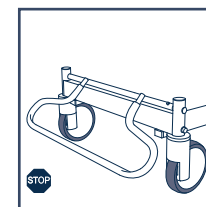

Brake

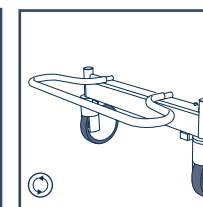

Free movement

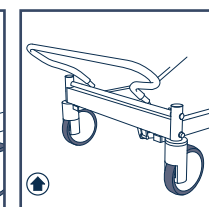

Steering or 5th wheel

CLEANING

Daily cleaning and disinfection with at least 1000ppm concentration of chlorine solutions:

- Rails, headboard, footboard.

Cleaning and disinfection after a patient leaving or before a transfer:

- Mattress platform area, chassis of the bed.

Deep cleaning and disinfection every two months:

- Support and lifting area of the bed.

MAINTENANCE

Always check the correct state of:

- Electrical wiring
- Wheels.
- Braking mechanisms.


For more information, please consult the use and maintenance manual.




CONTROLS

INSTRUCTIONS OF USE


A	Rise backrest	K	Locking movements	U	Low position indicator led
B	Lower backrest	L	Indicator Led of locked movements	V	Trend / Reverse trend selector
C	Rise mattress platform	M	Flat Position (lower all sections simultaneously)	W	Battery indicator
D	Lower mattress platform	N	Cardiac chair position	X	Explorer Position (maximum height)
E	Rise legrest	O	Light under bed	Y	Emergency Trend position
F	Lower legrest	P	Emergency CPR	Z	Nursing call
G	Rise both simultaneously (<i>autocontour</i>)	Q	On/Off Button		
H	Lower both simultaneously (<i>autocontour</i>)	R	Safety exit position		
I	Reverse Trend	T	30° backrest indicator led		
J	Trend	U	Low position indicator led		

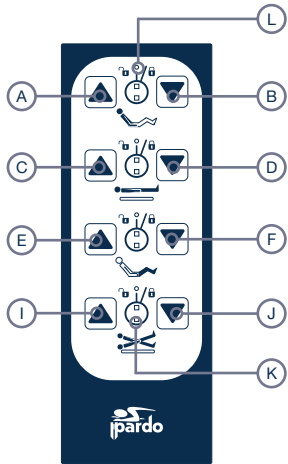


Patient hand control with locking

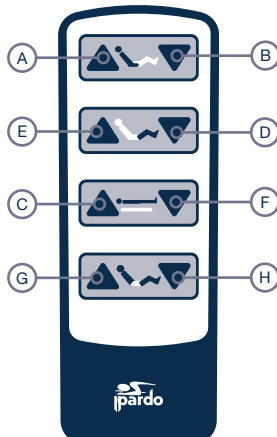


Patient hand control

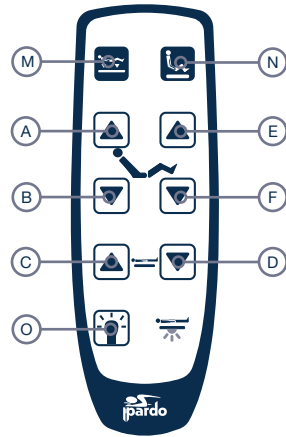


Patient hand control with light

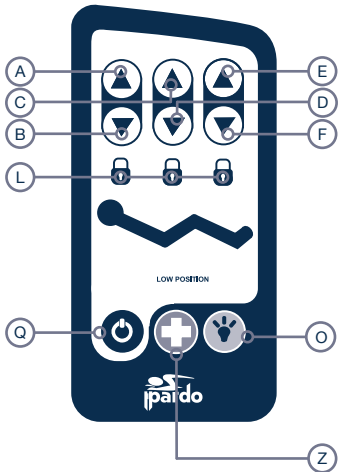


Membrane control

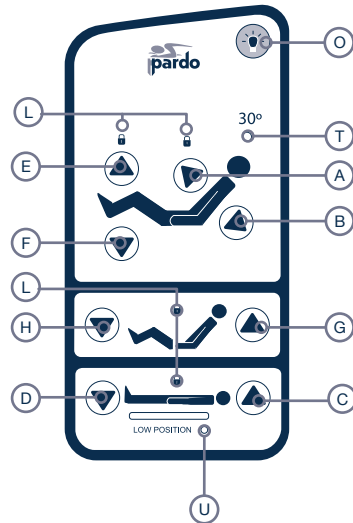




















Patient membranes

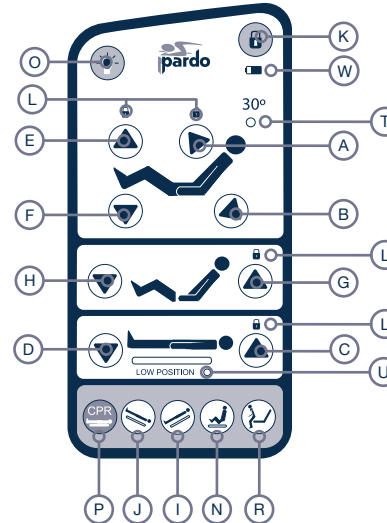

Nursing membranes

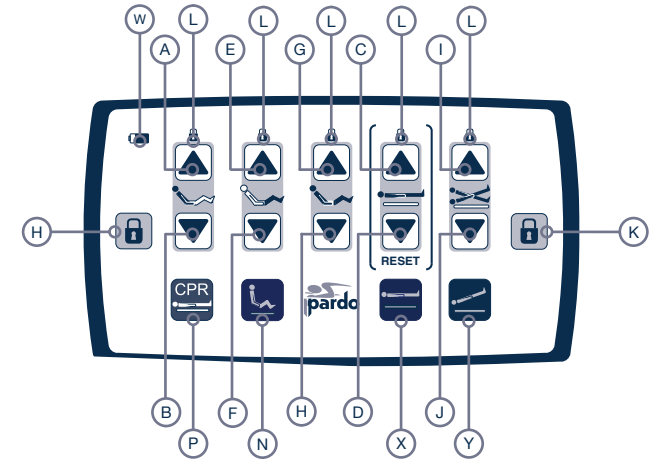

Nursing control

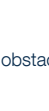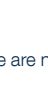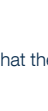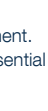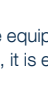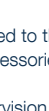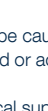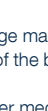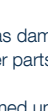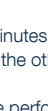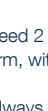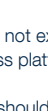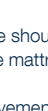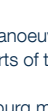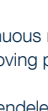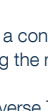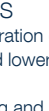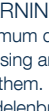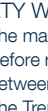

SAFETY WARNINGS

- The maximum duration of a continuous manoeuvre should not exceed 2 minutes, as damage may be caused to the equipment.
- Before raising and lowering the moving parts of the mattress platform, with the other parts of the bed or accessories, it is essential to check that there are no obstacles between them.
- The Trendelenburg and reverse Trendelenburg movement should always be performed under medical supervision.
- When the bed is in extended position, do not sit or lean on the foot part of the bed, since it may cause a bed dump.
- Do not use CPR bilateral actuating levers to raise the backrest of the bed, it may cause raise motor breakdown and makes a bed dysfunction.

