




Bora Kala Bed

QUICK REFERENCE SHEET

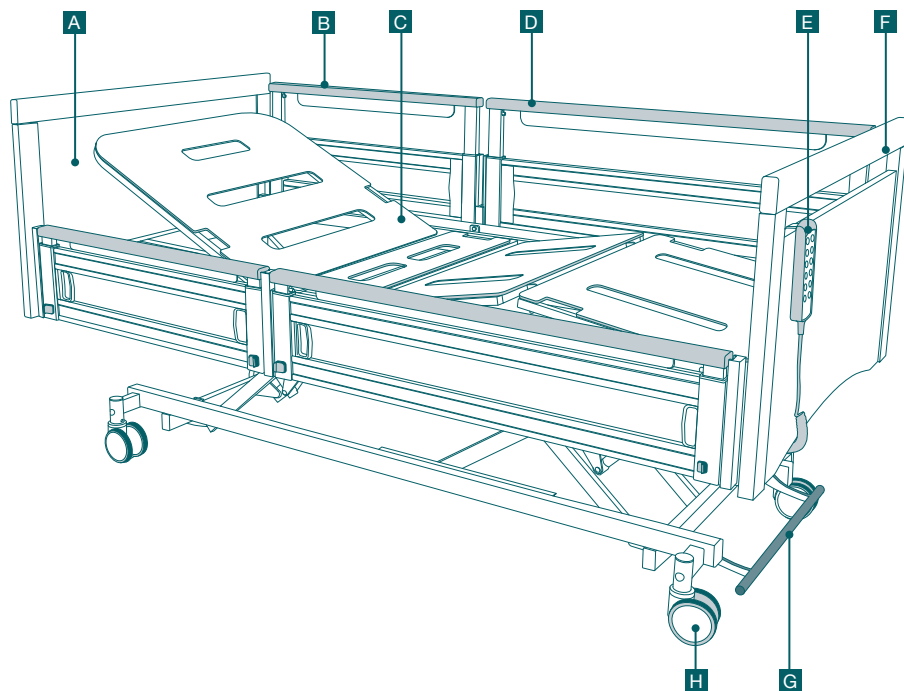

BED'S COMPONENTS

A	Screwed headboard	E	Patient hand control with locking
B	Short telescopic guardrail for backrest	F	Screwed footboard
C	HPL mattress platform	G	Centralised braking system
D	Long telescopic guardrail for legrest	H	Double wheels Ø10 cm.


www.pardo.es
 TECHNICAL SERVICE CENTRE
 (+34) 976 300 033

SIDE RAILS


Telescopic side rails:

- Rise first section: Pull the side rail up, from central section, until it is immobilized.
- Low first section: Slide both side handles inward simultaneously, without releasing said handles slide the side rail down.



- Rise handrail: Simultaneously press the two devices placed on the side rails and at the same time slide the railing up, until locking.
- Low handrail: Simultaneously press the two devices placed on the side rails and at the same time slide the railing down.

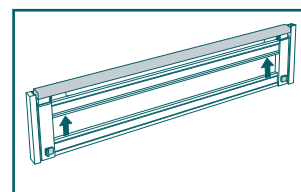


Rise first section

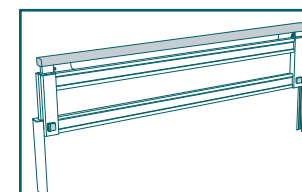

Lower first section

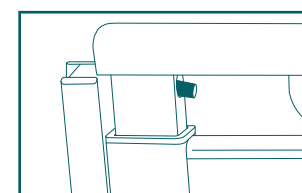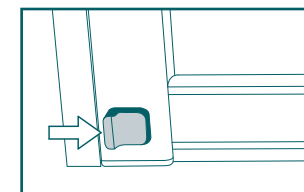

Rise and lower handrail

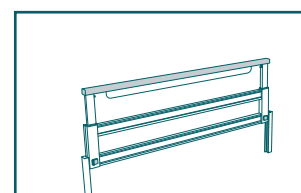

CLEANING

- Daily cleaning and disinfection with soapy water:
- Rails. Head board. Foot board.
- Cleaning and disinfection after a patient leaves or before a transfer:
- Mattress platform area. Chassis of the bed.
- Thorough cleaning and disinfection every two months:
- Support and lifting area of the bed.

MAINTENANCE

- Always check the correct state of:
- Electrical wiring
 - Wheels.
 - Braking mechanisms.

For more information, please consult the use and maintenance manual.




OTHER FUNCTIONS

Centralised braking system (1):


- Braking the bed: Pedal in low position.
- Moving the bed: Pedal in intermediate position.
- Steering wheel: Pedal in raised position.

Extended bed base (2):


- Pull and turn the locking knobs 180° to unlocked the anchoring system.
- Pull the footboard until extended the bed base completely.
- Push until listening "click".

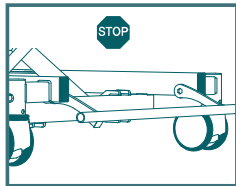




Brake

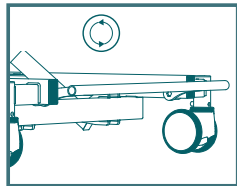

Free movement

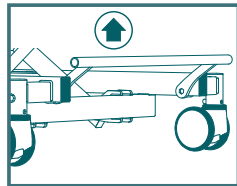

Steering wheel

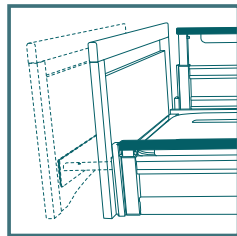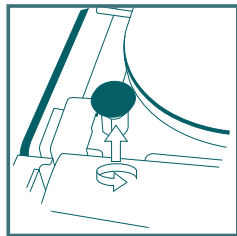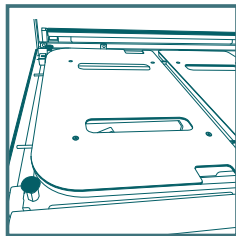

Extract the extended bed base


LOCK AND UNLOCK OF PATIENT CONTROL

Function locking is used to disable bed basic functions, mainly to lock movements which has to be done under medical guidance.


To Lock and Unlock the bed functions, only extract or insert the locking key. This key is localised at the bottom of the wired control.

- **Unlock:** Insert the locking key on the lateral of wired control. The Indicators (L) are lighted on green colour.
- **Lock:** Extract the locking key on the lateral of wired control. The Indicators (L) are lighted on red colour.


CONTROLS

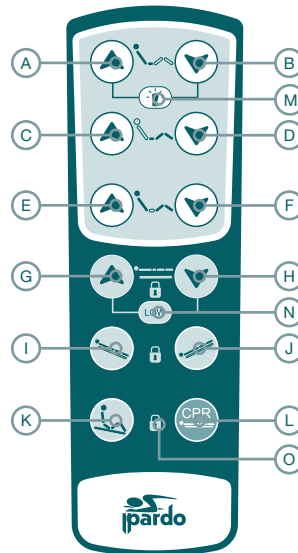

Patient hand control with locking

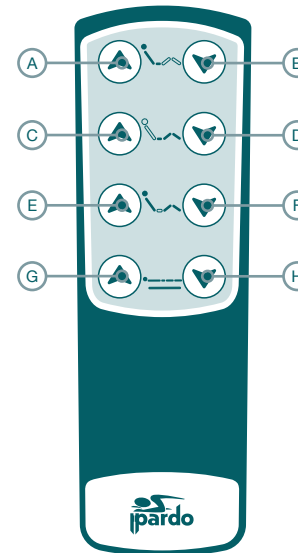

Patient hand control without locking

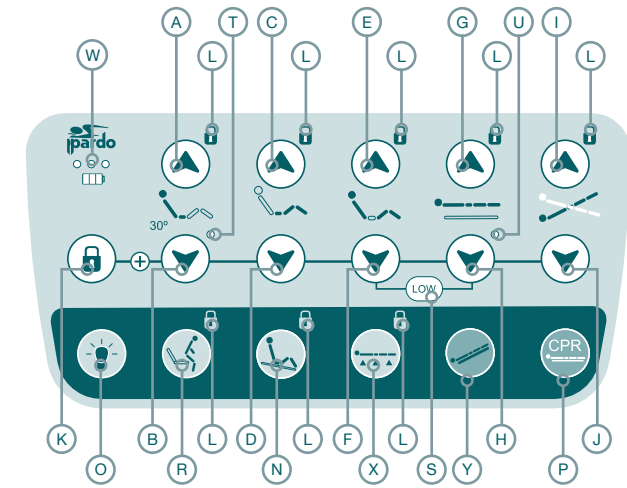

Wired nursing control with locking

INSTRUCTIONS OF USE

A	Rise backrest	I	Reverse trendelenburg	S	Low Position (minimum height) (2)
B	Lower backrest	J	Trendelenburg	T	30° tilting backrest indicator led
C	Rise legrest	K	Locking movements	U	Low position indicator led
D	Lower legrest	L	Indicator Led of locked movements	W	Battery indicator led
E	Rise all sections simultaneously (<i>autocontour</i>)	N	Cardiac Chair position	X	Explored position (maximum height)
F	Lower all sections simultaneously (<i>autocontour</i>)	O	Light under bed (1)	Y	Emergency trendelenburg position
G	Rise mattress platform	P	Emergency CPR position		
H	Lower mattress platform	R	Safety exit position		

(1) Press simultaneity A+B for several seconds

(2) Press simultaneity G+H, when the mattress platform is on minimum height.



SAFETY WARNINGS

- The maximum duration of a continuous manoeuvre should not exceed 2 minutes, as damage may be caused to the equipment.
- Before raising and lowering the moving parts of the mattress platform, with the other parts of the bed or accessories, it is essential to check that there are no obstacles between them.
- The Trendelenburg and reverse Trendelenburg movement should always be performed under medical supervision.
- When the bed is in extended position, do not sit or lean on the foot part of the bed, since it may cause a bed dump.

